

Release Notes MobiMed Smart v4.2.21

Software Article Number 490211030 (Software Build No Release-4.2.21.0_17.04.21.3)

Customer specific changes

- SWASFT CCN SW071 - ePCR Config Changes (Air Amb, Handover) (7263)
 - Implement ePR changes (7873)
 - Add ECG for all "Search archive" reports (7875)
- SCASFT Add scroll bar to ECG tab to enable scrolling after resizing for on screen keyboard (7916)


General improvements

- Make it possible to install MSI packets based on computer model (7292)
- Implement navigation from Vital Signs Bar to corresponding VP page (clicking SpO2 in VSB shall open the SpO2 page if monitoring is active) (7888)
- Philips FR3 - Add button for syncing time to device (7904)
- AdminTool RemoteUpdate not listing licenses/units if duplicate license number has been entered (7931)
- Internal/Miscellaneous
 - Refactor Plugins to write time values with respect to the group time restrictions (7299)
 - Detect if connected care episode cannot be set twice (7375)
 - Move customer ePR configurations to a separate component that can be distributed independently (7887)
 - Improve error handling for ePR searches to easier detect heavy questions. (7912)
 - Change blood pressure measurement to show values for a Test Patient as soon as measurement is done (7927)

Fixes

- Incident and incident record tabs disappear when transferring leadership of an incident with multiple patients (7222)
- Improve program stability when network connection is changed (7380)
- Fix crash when opening another patient while having a signed patient open in monitor view (7386)
- HWS crash when using an invalid ECG export setting (7389)
- Prevent possibility to lookup history without PDS search (if PDS performed on a previous NHS number) (7392)
- Philips FR3 - Time in preview should be report creation time, not report import time (7878)
- MDT DispatchPlugin should set age/DoB in correct way (7886)
- CWS will only show a few events if session contains more than 1000 events with a -1 as end time (7903)
- Remove small gaps in real-time waveform drawing occurring on some screen resolutions (7907)
- Correct question order when reopening pathways, now displayed backwards (7915)
- Radio number dialog should not overwrite active ePR's region when not using settings tree hierarchies (7920)
- AdminTool, detailed view for unit info could show leftovers from old unit if no network connection (7921)

- CoroNetExport - ECG export shall use updated patient number and ID (7923)
- Ambulance Ledger: ETA timestamp should be presented in local time (7932)
- MU Icon - When connecting MU from the icon in MMM, the icons state is not updated (7937)
- Make sure MU connection dialog is frequently updated (7381)

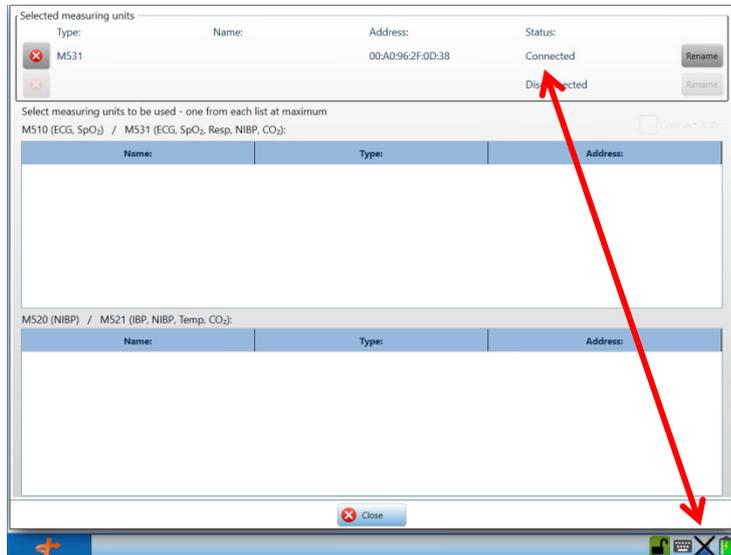

*Numbers in brackets are ID-numbers from Ortivus internal source control, work item tracking, reporting and project management tool.

Required ePR versions

The functionality in this release requires the below ePR versions.

- SCASFT : SCASFT_Release-4.2.21.0_17.04.21.3
- SWASFT: SWASFT_Release-4.2.21.0_17.04.21.3

Compatible hardware for ambulance units

The product is released for the following hardware platforms:

- Getac RX10H
- Panasonic H2

All versions of Measuring Units M531 and 300C are supported.



Release Information

This is a commercial release on the following markets:

- United Kingdom
- Sweden

This release replaces all previous 4.2 releases.

Upgrade to 4.2.21 can be done from all previous 4.2 releases.